# **PERIODONTAL** TREATMENT GUIDE

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Lang, Lindhe, Clinical Periodontology and Implant Dentistry (5. Ausg.) 2008(2) 655-673. Lang et al., Qualitätssicherung in der Parodontologie. Interdisziplinäre Fortbildungswoche IWF zur Qualitätssicherung in der Zahnmedizin der Schweizerischen Zahnärzte-Gesellschaft SSO, 1999  
Saxer UP, Muhlemann HR, Motivation and education. SSO Schweiz Monatsschr Zahnheilkd 1975, 85, 905-919 Lang N, Tan WC, Krähenmann MA, Zwahlen M, A systematic review of the effects of full-mouth debridement with and without antiseptics in patients with chronic periodontitis. 6. Europäischer Workshop über Parodontologie, Feb. 2008, Ittingen, Thurgau, Schweiz Heitz Mayfield LJA, Trombelli L, Heitz F, Needleman I, Moles D, A systematic review of the effect of surgical debridement vs. non-surgical debridement for the treatment of chronic periodontitis. J Clin Periodontol 2002, 29 (3) 92-102 Herrera D, Alonso B, Leon R, Roldan S, Sanz M, Antimicrobial therapy in periodontitis: the use of systemic antimicrobials against the subgingival biofilm. 6. Europäischer Workshop über Parodontologie, Feb. 2008, Ittingen, Thurgau, Schweiz Gaunt F, Devine M, Pennington M, Vernazza C, Gwynett E, Steen N, Heasman P, The cost-effectiveness of supportive periodontal care for patients with chronic periodontitis. 6. Europäischer Workshop über Parodontologie, Feb. 2008, Ittingen, Thurgau, Schweiz Schwarz F, Aoki A, Becker J, Sculean A, Laser application in non-surgical periodontal therapy: a systematic review. 6. Europäischer Workshop über Parodontologie, Feb. 2008, Ittingen, Thurgau, Schweiz Claffey N, Nylund K, Kiger R, Garrett S, Egelberg J, Diagnostic predictability of scores of plaque, bleeding, suppuration and probing depth for probing attachment loss. 3.5 years of observation following initial periodontal therapy. J Clin Periodontol 1990, 17 (2) 108-114 Eberhard J, Jervoe-Storm PM, Needleman I, Worthington H, Jepsen S, Full-mouth treatment concepts for chronic periodontitis: a systematic review. J Clin Periodontol 2008, 35 (7) 591-604 Lang N et al., A systematic review of the effects of full-mouth debridement with and without antiseptics in patients with chronic periodontitis. J Clin Periodontol 2008, 35 (8) 8-21 Quirinen M, Mongardini C, de Soete M, The role of chlorhexidine in the one-stage full-mouth disinfection treatment of patients with advanced adult periodontitis. J Clin Periodontol 2000, 27 (8) 578-589 Swierkot K, Flores de Jacoby L, Mengel R et al., One-stage full-mouth disinfection versus quadrant and full-mouth root planing. J Clin Periodontol 2009, 36, (3) 240-249 Quirinen M et al., Benefit of „one-stage full-mouth disinfection“ is explained by disinfection and root planing within 24 hours: a randomized controlled trial. J Periodontol, 33, (9) 639-647 Wang D et al., Antibody response after single-visit full-mouth ultrasonic debridement versus quadrant-wise therapy. J Clin Periodontol 2006, 9, 632-638 Lindhe J et al., Special issue of the Journal of Clinical Periodontology on enamel matrix proteins. J Clin Periodontol 1997 Sept. 24(9) Bosshardt D et al., Effects of enamel matrix proteins on tissue formation along the roots of human teeth. J Periodontol. res 2004, 40, 158 Heijl, Heden et al., Enamel matrix derivative (Straumann® Emdogain) in the treatment of intrabony periodontal defects. J Clin Periodontology 1997; 24; 705-714 Pontoriero et al., The use of barrier membranes and enamel matrix proteins in the treatment of angular bone defects. J Clin Periodontol. 1999; 26(12): 833-40 Heden, Wennström et al., Five-Year Follow-up of Regenerative Periodontal Therapy with Enamel Matrix Derivative at Sites with Angular Bone Defects. J Periodontol 2006; 295-301 Sculean et al., Treatment of Intrabony Defects With an Enamel Matrix Protein Derivative or Bioabsorbable Membrane: a 8-Year Follow-up Split-Mouth Study. J Periodontol 2006; 77(11), 1879-1886 McGuire MK, Nunn M, Evaluation of Human recession defect treated with coronally advanced flaps and either Enamel Matrix Derivative or Connective Tissue. J Periodontol 2003; 74: 1110-1125 McGuire MK, Cochran DL, Evaluation of Human recession defect treated with coronally advanced flaps and either Enamel Matrix Derivative or Connective Tissue. J Periodontol 2003; 74: 1126-1135 Cueva MA, Boltchi FE, Nunn ME, Rivera-Hidalgo F, Rees T, A comparative study of coronally advanced flaps with and without the addition of enamel matrix derivative in the treatment of marginal tissue recession. J Periodontol 2004 Juli. 75(7):949-56 Castellanos A, de la Rosa M, de la Garza M, Caffesse RG, Enamel matrix derivative and coronal flaps to cover marginal tissue recessions. J Periodontol. 2006 Jan. 77(1):714 Jepsen, Meyle et al., A randomized clinical trial comparing enamel matrix derivative and membrane treatment of buccal Class II furcation involvement in mandibular molars. Part I: Study design and results for primary outcomes. Part II: secondary outcomes. J Periodontol. 2004, Aug. 75(8): 1150-60 Meyle, Jepsen et al., A randomized clinical trial comparing enamel matrix derivative and membrane treatment of buccal Class II furcation involvement in mandibular molars. Part i: Study design and results for primary outcomes. Part ii: secondary outcomes. J Periodontol. 2004, Sept. 75(9): 1188-95 Lindhe J et al., Special issue of the Journal of Clinical Periodontology on Enamel Matrix Proteins. J Clin Periodontol 1997 Sept. 24(9) Bosshardt D et al., Effects of enamel matrix proteins on tissue formation along the roots of human teeth. J Periodontol. res 2004, 40, 158 Cortellini P, Tonetti MS, microsurgical approach to periodontal regeneration. initial evaluation in a case cohort. J Periodontol 2001,72, 559-569 De Sanctis M, Zucchelli G, Clauser C, Bacterial colonization of barrier material and periodontal regeneration. J Clin Periodontol 1996, 23, 1039-1046 Esposito M, Grusovin MG, Coulthard P, Worthington HV, Enamel matrix derivative (Emdogain) for periodontal tissue regeneration in intrabony defects. Cochrane Datenbank Syst. Rev. 2005, CD003875

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© Institut Straumann AG, 2011. Alle Rechte vorbehalten.